smart D-Dimer Test

Testkit for quantitative determination of D-Dimer in plasma on smart Photometer.

EUROLyser
DIAGNOSTICA

Eurolyser Diagnostica GmbH Tel. +43 (0) 662 43 21 00
Bayernstraße 11a Fax. +43 (0) 662 43 21 00 50
5020 Salzburg/Austria www.eurolyser.com

Order Information	Indication	Kit size
Order No.: ST0150	smart D-Dimer-Test Kit	16 tests/kit
Order No.: ST1500	smart D-Dimer-Control Kit	2 x 1 ml / high and low
Order No.: ST1510	smart D-Dimer-Calibrator Kit	1 x 1 ml

English:

Test kit preparation:
Allow single test min. 10 minutes to warm to room temperature by placing the test into the test kit rack. Put back test kit package into refrigerator.

Summary:

Elevated levels are found in patients with confirmed deep venous thrombosis (DVT), pulmonary embolism (PE), DIC, and trauma. D-Dimer levels rise during pregnancy and high levels are associated with complications.

Limitation and Interferences of the Test

Each measurement series should be checked with smart D-Dimer Control high and low. Presence of rheumatoid arthritis factor may result in false-positive results (influence not quantified). No interference is found from CRP (<50mg/L), bilirubin (<0.27 g/L), haemoglobin (<6.7g/L), or triglycerides (<3.3g/L). Results from patients with heterophilic antibody should be interpreted with caution since this test kit contains mouse antibodies and interference may occur resulting in falsely elevated or decreased values.

Methode Analytical Range:

A monoclonal anti D-Dimer antibody (mouse J1F-23) coupled to latex particles forms high molecular weight immunocomplexes with D-Dimer in the test sample. The resulting change in turbidity of the sample is detected photometrically by turbidimetric analysers.

Analytical Range: 85 - 3000 ng/ml (LOT specific)

Principle

Kinetic measurement of the D-Dimer concentration at 700nm (absorbance)

Testkit:

ERS Cuvette filled with 900µl buffer
ERS Cap filled with 220µl antibodies (mouse J1F-23) latex reagent

Stability and Storage:

Store at 2-8°C. DO NOT FREEZE. Kit can be used until expiry date printed on the labels.

Warning:

This testkit is for in vitro diagnostic use only. Do NOT INGEST. Avoid contact with skin and eyes. Contains sodium azide, which may react with lead or copper plumbin to form explosive compounds.

Sample Material:

Preparation of plasma samples
Plasma separation: Mix 9 parts of venous blood and 1 part of Sodium Citrate Solution (0.11 mol/L) and centrifuge for 15 minutes at a RCF of at least 2500 g (corresponding to DIN 58905).
The samples should be used as fresh as possible or should be stored before usage at -20°C.

Reference Ranges:

Samples with a D-Dimer concentration > 250 ng/mL are considered as pathologic.
The probability of absence of thrombotic events is high if the concentration values are < 135 ng/mL.
Samples above 3000 ng/mL should be diluted with Saline Solution and re-assayed.

Control Material:

For internal quality assurance the smart D-Dimer Control Kit should be used; D-Dimer Control Kit Order No.: ST1500

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