## smart $\mu$ Alb Test

Testkit for quantitative determination of albumin in urine by smart Photometer.

**EURO**Lyser  
DIAGNOSTICA

Eurolyser Diagnostica GmbH Tel. +43 (0) 662 43 21 00  
Bayernstraße 11a Fax. +43 (0) 662 43 21 00 50  
5020 Salzburg/Austria www.eurolyser.com

### Order Information

Order No.: ST0130  
Order No.: ST1300  
Order No.: ST1310

### Indication

smart  $\mu$ Alb-Test Kit  
smart  $\mu$ Alb-Control Kit  
smart  $\mu$ Alb-Calibrator Kit

### Kit size

32 tests/pack  
1 x 1-2 ml  
1 x 1 ml

### English:

#### Test kit preparation:

Allow single test min. 10 minutes to warm to room temperature by placing the test into the test kit rack. Put back test kit package into refrigerator.

### Introduction:

A small amount of protein is excreted daily into the urine of healthy individuals. The extracted proteins are mucoproteins, most of which are filtered out of the uriniferous tubules and the glomeruli. Albumin, a protein of molecular weight of 50000 is not easily filtered out and is excreted into the urine (microalbuminuria). This makes albumin excretion into the urine a useful indicator of early glomerular disease. Microalbuminuria is a condition characterized by increased urinary excretion of albumin in the absence of overt nephropathy. Microalbumin has been reported in several studies to predict development of diabetic nephropathy and its mortality risk in diabetic patients. Because microalbuminuria may be reversible if diabetes is well controlled, the early detection of Microalbumin may be very beneficial in treatment programs for diabetes.

### Principle:

When a sample is mixed with anti human albumin goat antiserum, agglutination is caused by the antigen antibody reaction. the turbidity is measured with SAMRT340 and albumin in the sample is quantitatively determined

### Testkit:

ERS Cuvette filled with Buffer Reagent pH7,6 100mM Tris  
ERS CAP filled with Antiserum Reagent pH 7,6 20% Anti Human albumin goat antiserum 100mM Tris

### Storage and handling:

Kit should be stored at 2-8°C. Kit is stable until expiry date printed on the label

### Warnings and precautions:

For IN VITRO DIAGNOSTIC USE. Not to be used internally in humans or animals. Normal precautions exercised in handling laboratory reagents should be followed. Reagent in kit contains sodium azide as a preservative. Sodium Azide may form explosive compounds in metal drain lines.

### Dispose:

Dispose reagent kit according to your local requirements.

### Specimen collection and preparation:

The specimen should be fresh or a 24 hour urine. Urine specimen should be stored refrigerated 2-8°C  
The specimen may be stored refrigerated up to 2 weeks.

### Expected values:

The expected value for Microalbumin is < 15mg/l or 30-300mg/24 hours. Each laboratory should establish its own expected values using this kit.

### Control material:

A quality control program is recommended for all clinical testing laboratories. A control material is offered with order no ST1300

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#### 2.) Application with laboratory pipette:

- 2.1a) Aspirate 20 $\mu$ l urine with laboratory pipette
- 2.1b) Aspirate 20 $\mu$ l urine with transfer pipette (see "Correct usage of the transfer pipette")
- 2.2) Pipette 20 $\mu$ l urine into ERS cuvette
- 2.3) Apply ERS Cap

#### 3.) Sample processing with smart photometer:

Please follow the guidelines in the users manual

- 3.1) Enter Name and ID
- 3.2) Insert ERS Cartridge and press START ►

#### Correct usage of the transfer pipette:

- 1.) Firmly squeeze and hold upper bulb.
- 2.) Completely immerse tip into the liquid of lysing cup.
- 3.) Release bulb slowly to aspirate liquid. Overflow of liquid is not critical as it flows into the chamber and won't be dispensed when pipetting!
- 4.) Put end of pipette into ERS cuvette
- 5.) Squeeze upper bulb completely
- 6.) With the bulb squeezed, release the pipette from liquid